

# Polyshine

## Permanent brightener for acrylic, resin and flex material.

### DESCRIPTION

100% natural and NOT harmful for the environment, produces a better finish due its unique and uniform abrasive properties.

Because its composition and abrasive characteristics, it does easier and faster the work.

### Specifications

Appearance	Cream
Color	Pink
Solubility	Insoluble
Moisture content	< 2.0%
pH (10% sol'n acuosa)	8.0 a 11.0

### INSTRUCTIONS

To obtain a better application put a small quantity that barely cover the Surface to bright, let stand for a few seconds to obtain a better glossy finish in acrylic, resin and flex materials.

### Advantages.

A faster work.

Polishine doesn't keep odors, this characteristic makes this product unique from other traditional polish.

### Presentation

Tube containing 100 gr

## Storage.

This material has a lifespan of 5 years from the manufacturer date, always remain in correct storage conditions, free of pollution, in an environment protected from sunshine, moisture and in its original packaging.

**SECURITY**

*Avoid contact with the eyes. If this occurs, wash with plenty of water. If irritation persist, seek medical advice.*

**Manufacturera Dental Continental, S.A. de C.V.**  
**Industria del Plástico 2113 Fracc. Zapopan Industrial Norte Zapopan Jalisco México**  
**Tel.: 01 33 3633 7402 [www.dental.com.mx](http://www.dental.com.mx)**

Ultima Revisión Mayo 2022  
Reviso/Autorizo APE  
Aseg. De Calidad D y D  
Rev. No. 0

**Australian Distributor:** Ultimate Dental Supply  
660a South Rd  
Moorabbin Vic 3189  
Tel: 03 9532 1799  
Email: [info@ultimatedental.com.au](mailto:info@ultimatedental.com.au)

**Emergency Phone No:** Poisons Hotline (24 hours/7days) 13 11 26